A SWISS BRAND

ELECTRIC BREAST PUMP

MODEL GM-30

USER'S MANUAL

dr-frei.com

EN ELECTRIC BREAST PUMP

Model GM-30

User's manual (4-10)

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**ELECTRIC BREAST PUMP
MODEL GM-30
USER'S MANUAL**

Thank you for choosing **Dr. Frei**[®] Electric Breast Pump GM-30. We are sure that having appraised worthily the high quality and reliability of this device you will become a regular user of the products of **Dr. Frei**[®] Trade Mark.

Before starting to use the Breast Pump please study the instruction book carefully. Following the rules stated in the user's manual will provide for the faultless operation of the Breast Pump for a long period of time.

TABLE OF CONTENTS

ABOUT BREAST PUMP	5
Caution	5
Gentle Reminder	5
Component Diagram	6
USAGE	7
Eliminating Methods Of Lacking Suction	7
INSTALLATION METHOD	8
CLEANING	9
STERILIZATION	10
Boiling Sterilization	10
Pharmaceuticals Sterilization	10
Microwave Sterilization	10
WARRANTY	10

ABOUT BREAST PUMP

Breast feeding is the best choice for babies' growth. Newborn can obtain immunogenic molecules from breast milk, but sometimes you find it is inconvenient to feed your baby directly. Our breast pump is the ideal appliance which can imitate rhythm of baby's sucking, and its soft silica gel can give your breast a massage, alleviate pain and avoid the mam-milla collapses. Our breast pump makes nursing mothers healthier and more beautiful.

ATTENTION: All the safe display legends are contained in the operation instruction. Please operate the product after understanding symbols fully.

Caution

It is possible to hurt someone or something lightly by wrong using

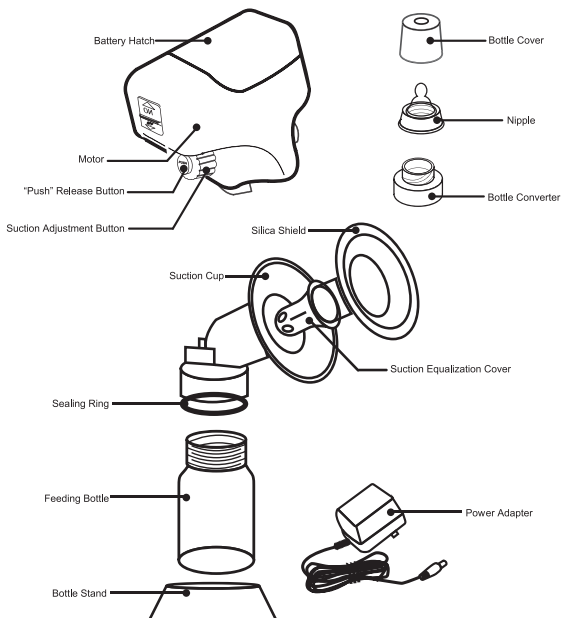
- Do not apply to other purpose.
- Do not close to fire.
- Install every component correctly, any improper or loose installation can make breast sucking abnormal.
- Please stop sucking immediately and consult doctor when you feel unusual of mam-milla or breast.
- It may cause pain by continuous sucking or overuse.
- Please stop using when feeling mam-milla or breast painful.
- Wash your hands before sucking.
- Do not put breast pump or disassembled component in the place where the children can easily reach it to avoid baby's hurting or eating.
- Do not drop and impact it strongly to avoid deformation or damage.
- Do not take out the breast pump after sterilization to avoid scald.
- Do not over-boil to prevent it aging premature.

Gentle Reminder

Breast pump is not the device for milk opening. The most important thing is that it can help Moms to relieve distress of swelling milk and prolonged breast-feeding time. The premise of usage is the adequate breast milk and unimpeded breast. Blocked breast (some is


caused by dent mammilla) or milk shortage is often existed among some new Moms, so you are proposed to have a professional examination for unimpeded breast before buying breast pump rather than make breast pump as milk opening device blindly. In fact, its suction is normal; most returned goods are caused by knowing products too little.

Component Diagram


* The specifications and the outer design are subject to change for improvement reasons without previous notice.

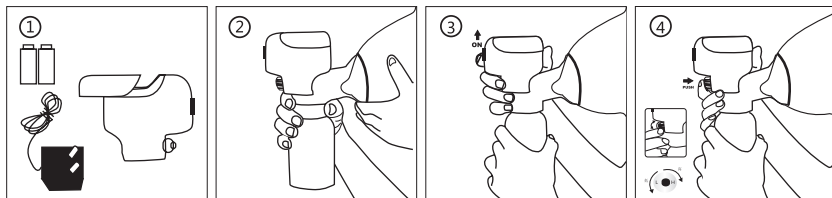- Please wash your hands before using.
- Please install the breast pump correctly after demolition, cleaning and sterilization. (Reference to demolition, cleaning, disinfection and installation instructions)
- Put batteries in Battery hatch or connect power adapter before using. (Figure 1)
- Please sit comfortably and open the dustproof cover for breast sucking.
- Please make your mammilla center aligned and let breast close to the breast shield to prevent air leakage. It will be close to breast more proper with one hand below your breast. (Figure 2)
- Connect the power supply, turn on the switch. (Figure 3)
- Adjust suction by turn the switch and release suction by press release button with proper rhythm. (Figure 4)

Eliminating Methods Of Lacking Suction

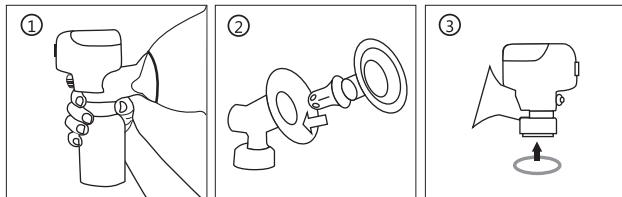- Confirm whether the suction cup is surrounding the breast or not. (Figure 1)
- Check the suction cup is be fixed with suction





equalization cover, silica shield and suction tegument well or not. (Figure 2)





- Check sealing ring is in proper position or not. (Figure 3)
- Inspect the power adapter is connected smoothly or not.
- Make sure that the power line does not be damaged or battery has enough quantity of electricity.
- Please let motor working for 30 seconds when the temperature is under 15 °C.

ATTENTION

- Please stop using when feeling uncomfortable or painful.
- Do not sucking by force if the breast milk still can not be out by using breast pump.
- Led breast pump may cause liquid leakage when there is breast milk in feeding bottle.

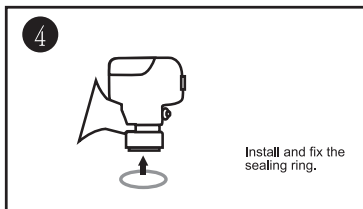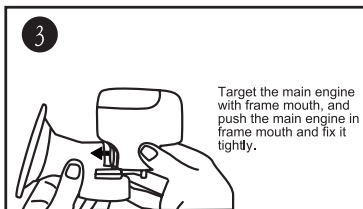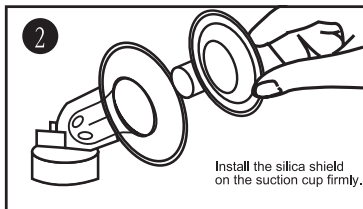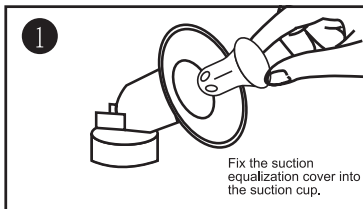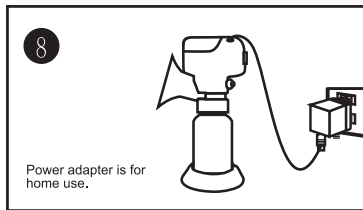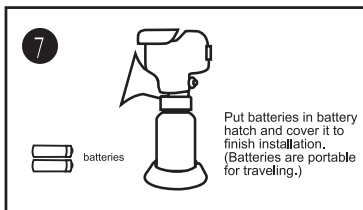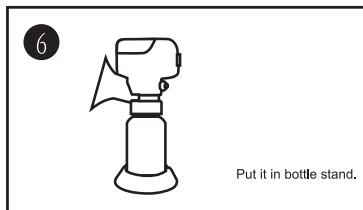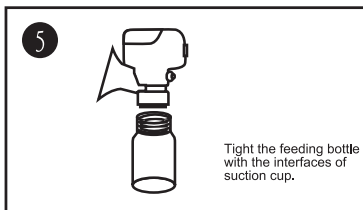INSTALLATION METHOD

CLEANING

- Please be sure to clean and sterilize before using
- Please wash it in warm water after using to prevent infections which caused by bacteria.
- Please clean it with soft sponge and special munificent. Do not wipe with decontamination powder brush and other hard objects which can easily produce scratches, white turbid and make the product dirty.
- Please wash all silica gel components (silica shield, sucking silica, vacuum film, silicon seal with soft sponge.
- Do not wash, sterilize and breach motor, or breast pump will be damaged.
- Please wipe the breast pump with a damp cloth.

STERILIZATION

Boiling Sterilization

- Please avoid scalding when sterilization
- Please sterilize it in a large pot with a large number of water to avoid touching to the bottom of pan.
- Please control the sterilization time in 2-3 minutes, or it will be injured earlier.
- Please remove the product after cooling. You may scald and hurt under high temperature and deformation by overheating.
- It is better to Sterilize with our sterilizer

Pharmaceuticals Sterilization

- Take apart component and sterilize bottle & nipples with liquid antiseptic.
- Please sterilize according to instruction of liquid antiseptic. The painting color may become lighter by pharmaceuticals sterilization

Microwave Sterilization

- Take apart component and sterilize according to instruction of microwave oven & electronic microwave sterilizer
- Do not put breast pump into electronic microwave sterilizer directly for sterilization

WARRANTY

This warranty applies to the electric breast pump model GM-30 and lasts for 1 year from the date of purchase. Warranty applies to electric part of the device only. The warranty service does not apply to damages caused by: mechanical damages, damages from natural disasters, attempt of self-maintained alterations made to the device, alter or repair of the device and improper handling.

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Manufactured by the order of: FreiCare Swiss GmbH,
■ Zhejiang Huijun Infant And Child Articles Co., Ltd. No.1 Sheng'an Road, Shangsheng