A SWISS BRAND

MANUAL BREAST PUMP

MODEL GM-10

USER'S MANUAL

dr-frei.com

(EN) MANUAL BREAST PUMP

Model GM-10

User's manual (4-8)

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MANUAL BREAST PUMP
MODEL GM-10
USER'S MANUAL

Thank you for choosing **Dr. Frei®** Manual Breast Pump GM-10. We are sure that having appraised worthily the high quality and reliability of this device you will become a regular user of the products of **Dr. Frei®** Trade Mark.


Before starting to use the Breast Pump please study the instruction book carefully. Following the rules stated in the user's manual will provide for the faultless operation of the Breast Pump for a long period of time.

TABLE OF CONTENTS

Components List.....	5
HOW TO ASSEMBLE MANUAL BREAST PUMP	5
HOW TO USE	6
Notice	6
HINTS	7
Breast Milk Storage	7
Warm Tips	8

Components List

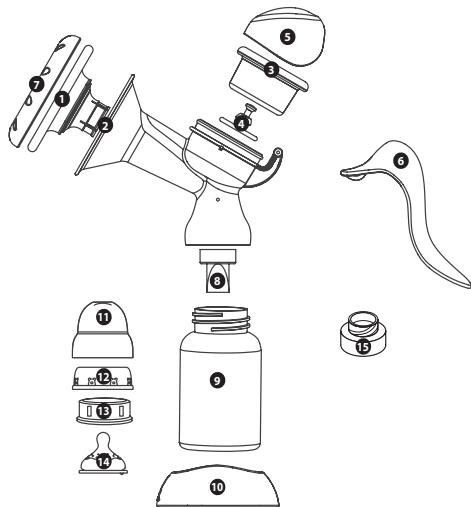1. Soft breast shield
2. Funnel unit
3. Diaphragm
4. Stem
5. Cap
6. Handle
7. Dustproof cover
8. Valve Membrane
9. Milk bottle
10. Bottle stand
11. Bottle cap
12. Bottle ring
13. Sealing cap
14. Teat
15. Bottle converter

* The specifications and the outer design are subject to change for improvement reasons without previous notice.

HOW TO ASSEMBLE MANUAL BREAST PUMP

1. Please insert the stem (4) into the silicone diaphragm (3).
2. Insert the silicone diaphragm (3) into the funnel unit (2) as shown. Ensure these two parts are fitted together.
3. Insert the silicone breast shield (1) by aligning the contours of shield to that of the funnel unit (2).
4. Slide the stem (4) into the fork of handle (6), and then assemble the handle on the pivot of funnel unit (2).

5. Assemble the dustproof cover (5) to be locked on the funnel unit (2).
6. Align the cover (7) over the funnel.
7. Push the valve membrane (8) into the pump unit (2).
8. Screw the bottle (9) onto the assembled funnel unit (2).
9. Place the assembled bottle (9) onto the bottle stand (10).

* Ensure parts are assembled correctly.

HOW TO USE

1. Clean all the parts of breast pump, using the steam sterilizer, wash your hand and assemble all the parts properly, tightened to avoid leakage.
2. Before start sucking milk, gently massage the breast or use a warm towel to compress the breast, which is helpful for extracting breast milk.
3. Sit on a chair relaxed, and tilt your body forward slightly.
4. Press the breast shield gently and tightly against your breast to create a proper seal.
5. Press the handle gently and feel the suction. You can easily control the suction via the fingertip action on the handle.
6. Press the handle for 2 or 3 seconds at a time to imitate the babies natural sucking motion.
7. Sucking two breasts alternately, no more than 10 minutes each time.

PLEASE NOTE: If you can not draw any milk from the breast after five minutes of operation, discontinue use and try again later.

Notice

- All parts should be cleaned and sterilized using the steam sterilizer. Keep it dry and sealed.
- Do NOT use the device when it is not attached to the bottle because of possible damage.
- Do NOT put in direct sunlight.
- If you have any discomfort or painful feeling after use, consult your Physician or Lactation Consultant.
- This product is for single use only, if used by more than one person may cause a health risk.

When Should You Use The Breast Pump?

1. When there is insufficient Breast Milk Flow, use the Breast Pump to simulate the Babies Suction to promote milk flow.
2. When there is too much Breast Milk, use the Breast Milk Pump to suck away excess milk.
3. In the case of Bulgy Breast Pain, use the Breast Pump to comfort yourself.
4. Use the Breast Milk Pump to draw milk for feeding at a later time.

How To Use The Breast Milk Pump Most Effectively?

1. Choose a time when you are least likely to be disturbed.
2. Having a picture of your baby with you during use helps promote milk excreting.
3. Before Using, Massage your breast gently or apply a warm towel to your breast for a few minutes to help excrete the milk.
4. Being relaxed and warm are helpful hints for excreting breast milk; you can try using it after taking a shower.
5. It is suggested to use the Breast Pump when nursing the baby on the other breast.

Breast Milk Storage

If the breast milk would not be used for feeding baby immediately, it must be stored in the feeding bottle and preserved by freezing in the refrigerator. If breast milk to be stored more than 48 hours, It must be rapidly frozen, and use of dedicated storage containers, DO NOT just use a feeding bottle. If you need long time storage of breast milk, we recommend consulting your doctors or breast-feeding professionals, they will provide specialized storage ways.

Warm Tips

Modern people know that Breast Feeding has many unique advantages and adds to the babies healthy development. The devise can be used to extend feeding time and or used for later feeding by the working Mother. We sincerely hope you utilize the Manual Breast Pump to its full advantage, and make nursing your child easier and more pleasant.

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Manufactured by the order of: FreiCare Swiss GmbH,
■ Zhejiang Huiyun Infant And Child Articles Co., Ltd. No.1 Sheng'an Road, Shangsheng